


3M™ Scotchcast™ Quick Step Splint Application Guide

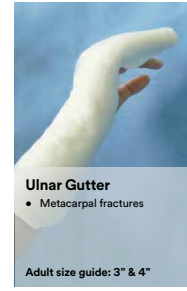

Ulnar Gutter
• Metacarpal fractures
Adult size guide: 3" & 4"



Extended Volar Slab
• Hand fractures
• Soft tissue injury
• Night resting splint
Adult size guide: 3" & 4"



Wrist & Thumb Splint
• Wrist fractures
• Scaphoid fractures
• Hand & thumb resting splint
Adult size guide: 3" & 4"



Sugar Tong Splint
• Forearm fractures
• Wrist fractures
Adult size guide: 3" & 4"



Thumb Spica
• Thumb fractures
• Thumb injuries
Adult size guide: 3"



Short Volar Slab
• Wrist fractures
• Wrist sprains
• Night resting splint
• Carpal tunnel
Adult size guide: 3" & 4"



Dorsal Splint
• Wrist fractures
• Wrist sprains
Adult size guide: 3" & 4"



Humeral U-Slab
• Humeral fractures
Adult size guide: 3" & 4"



Finger Splint
• Finger fractures
• Finger injuries
Adult size guide: 2" & 3"




Finger Splint
• Finger fractures
• Finger injuries
Adult size guide: 2" & 3"



Posterior Below Knee Splint
• Foot drop
• Foot fractures or injuries
• Ankle resting splint
• Sprains
Adult size guide: 3" & 4"




Below Knee Foot Drop Splint
• Foot drop
• Resting splint
Adult size guide: 3" & 4"

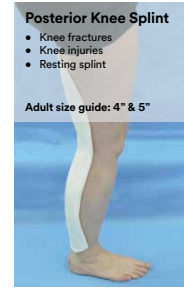

Posterior Knee Splint
• Knee fractures
• Knee injuries
• Resting splint
Adult size guide: 4" & 5"



Ankle Stirrup
• Ankle fractures
• Ankle sprains
Adult size guide: 3" & 4"

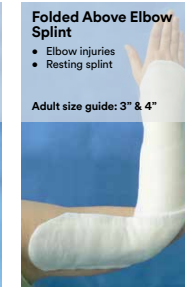

Folded Above Elbow Splint
• Elbow injuries
• Resting splint
Adult size guide: 3" & 4"



Butterfly cut for the ankle and elbow joints. Ensure the folds are on the outside of the splint and not in contact with the patient.



Example cut out for an extended volar slab including the fingers.



Example cut out over the elbow joint for a sugar tong and humeral u-slab.



Example cut out for a below elbow dorsal or short volar slab.



Example cut outs for a hand & thumb resting splint. This is achieved by cutting two separate pieces then joining together to create a thumb extension part as shown above.

3M Health Care Limited
Infection Prevention Division
3M House, Morley Street
Loughborough LE11 1EP
0845 873 4164
www.3M.co.uk/healthcare

3M Ireland Limited
The Iveagh Building,
The Park, Carrickmines
Dublin 18
00 353 1 280 3555
www.3M.ie

J309967
3M and Scotchcast are trademarks of the 3M company.
© 3M 2015. All rights reserved.